HeartStart MRx CO₂/EtCO₂ Measurement Addendum

English

This document is an addendum to the *HeartStart MRx Instructions for Use*. Keep this addendum with your *Instructions for Use*.

The HeartStart MRx EtCO₂ Option monitors carbon dioxide (CO₂) and measures end-tidal carbon dioxide (EtCO₂). The option provides an EtCO₂ value and a CO₂ waveform on the device's display.

Measurement Ranges

For devices using software versions F.02, R.02, 9.xx or any earlier versions, the HeartStart MRx does not measure or display EtCO₂ levels below 8 mmHg. For values below 8mmHg, you see a **- ? -** in place of the EtCO₂ reading in the parameter block and may see a valid, small CO₂ waveform on the display. The **- ? -** remains on the display until a measurement of 8mmHg or greater is obtained.

If you have a **- ? -** in the parameter block and no CO₂ waveform on the display, the waveform source is invalid. If invalid, check patient, confirm airway status and examine the FilterLine to make sure it is connected to the device and not kinked or pinched. For more information on the EtCO₂ Option see "Monitoring Carbon Dioxide" chapter in the *HeartStart MRx Instructions for Use*.

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Copyright © 2012